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PL

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INSTRUCTION MANUAL EN

ELECTRIC NASAL ASPIRATOR
ASPILOO

BEFORE USING THIS DEVICE, PLEASE READ THIS USER MANUAL! FOLLOW ALL SAFETY INSTRUCTIONS TO AVOID DAMAGE CAUSED BY IMPROPER USE! KEEP THIS USER MANUAL FOR FUTURE REFERENCE. IMPROPER USE CAN CAUSE PERSONAL INJURY AND ELECTRICAL DAMAGE!

SAFETY INSTRUCTIONS

- 1. Do not turn on the nasal aspirator while charging.
- 2. This device is intended for adult operation only.
- 3. Do not expose the device to high temperatures, high humidity, or direct sunlight.
- 4. The device is specifically designed for removing nasal mucus from a baby's nose. It should not be used for any other purpose!
- 5. Keep the parts and body clean and dry after each use.
- 6. Charging the battery to 100% every 2 months if don't use it for a long time.
- 7. Do not immerse the whole body in water during cleaning.
- 8. During use, the baby/child should not hold their breath, close their mouth, or swallow. Excessive vacuum pressure may cause an imbalance in ear pressure.
- 9. Do not exert excessive force when placing the suction tip into the nasal cavity.
- 10. If the baby/child experiences any discomfort while using the device, discontinue use immediately and consult a doctor.

OPERATION INSTRUCTION

	Power Switch: Long press to turn the device on/off. Short press to start/stop suction.
	Music Button: Short press to adjust the volume to levels 1 to 3. The button sequence is: 1-> 2-> 3-> Off-> On. Pressing again will switch to another music.
	Night Light Button: Power Off Mode: Long press to turn the night light on/off. Once turned on, single click to cycle through brightness levels 1 to 3. Power On Mode: After turning on the device, single click to cycle through brightness levels 1 to 3, and then turn off.
	Suction Power Decrease Button: After turning on the device, single click the „< „ button to decrease the suction power. You can select suction levels from 1 to 9.
	Suction Power Increase Button: After turning on the device, single click the „> „ button to increase the suction power. You can select suction levels from 1 to 9.

Note: Please note that before using a nasal aspirator, make sure to understand the characteristics of each nozzle and choose the appropriate one based on your child's age to ensure a comfortable and effective user experience.

Nozzle Type	Depth	Age Range
Conical Nozzle	Shallow	0-2 years old
Flat Nozzle	Deep	0 years and above
Angled Nozzle	Deepest	0 years and above

PRODUCT DESCRIPTION

Guide Before Use


- 1. It is recommended to disinfect the silicone nozzle and collection cup before use.
 - 2. Avoid using high-temperature devices such as microwaves for disinfection.
 - 3. This device is intended for adult operation only. Keep out of the reach of children.
 - 4. Before using, please check the product for proper sealing and ensure all accessories are complete.
 - 5. When using this device, please ensure that the child maintains a seated position with the nostrils facing downward.
- Avoid tilting the head back.

Product Features

- 1. Efficiently cleans baby's and children's nasal passages, ensuring safety and comfort.
- 2. Food-grade suction tip for safe and hygienic use, with convenient operation and maintenance.
- 3. Nine levels of suction strength to accommodate different needs.
- 4. Built-in soothing music and calming lights to help relax the baby during the cleaning process.
- 5. Long battery life for extended use.

Product Structure

Main Body

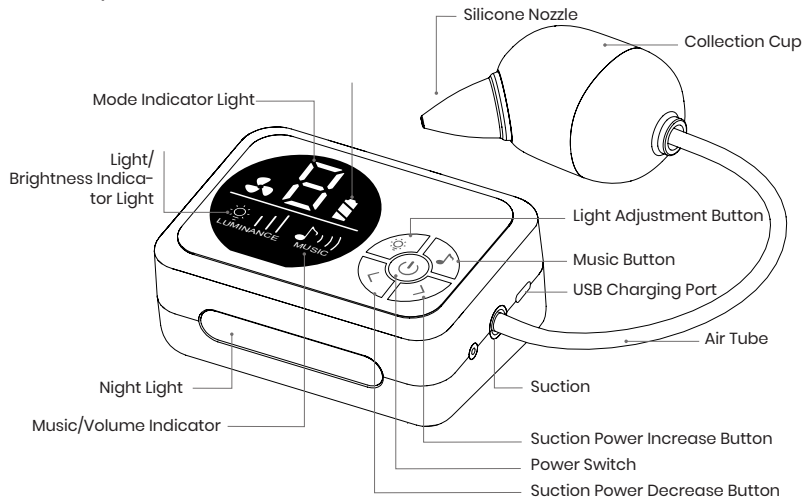Accessories

Collection cup lid


Diversion column

Collection cup bottom cover

Air Tube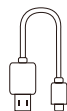

USB charging cable

Tweezers


Instruction manual

Conical nozzle


Flat nozzle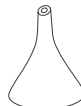


Angled nozzle

USER INSTRUCTIONS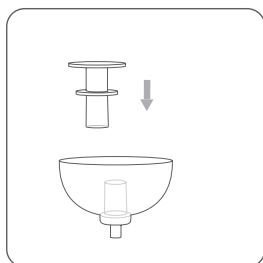


Step 1

Install the Diversion Column on the underside of the collection cup.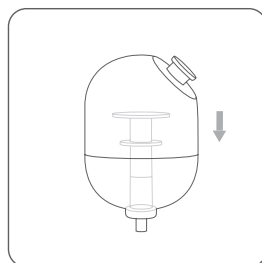


Step 2

Insert the collection cup onto the bottom cover.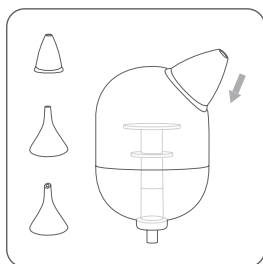


Step 3

Select the desired nozzle and install it onto the collection cup.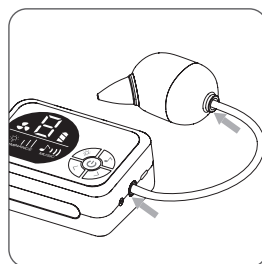


Step 4

Connect the collection cup to the main body's air intake port (Suction) using the tubing.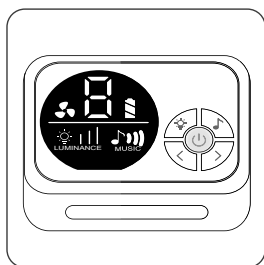


Step 5

Long press the power button to start the device (long press again to turn it off).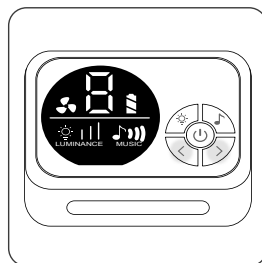

Step 6

Use the suction power adjustment button "<" or ">" to adjust the suction strength to the desired level.

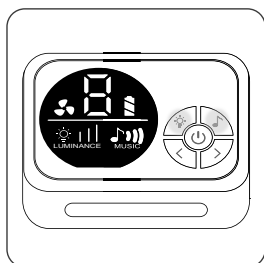
Step 7

Use the light button or the music button to activate a soothing mode (optional) that is suitable for your baby.

Step 8

Gently place the nozzle inside the baby's nasal cavity to effectively clean their mucus.

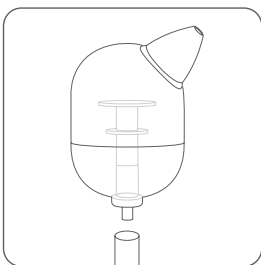
Step 9

After cleaning your baby's nasal mucus, remove the collection cup.

Step 10

After removing the collection cup components, rinse them promptly. (Note: The main body should not be washed with water!)

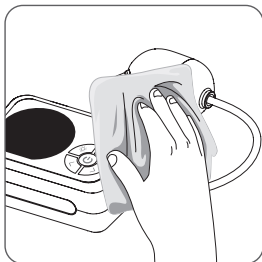Step 11

After cleaning, use a soft cloth to dry the surface of the device and store it properly in a cool place, avoiding direct sunlight.

Tips:

For better results, if the baby's nasal mucus is dry, it is recommended to use saline nasal spray to moisten it before using the electric nasal aspirator.

CLEANING AND STORAGE

1. Always turn off the power after use.
2. Remove the suction tip and collection cup and clean them with a cleaning agent and water.
3. Keep the parts and body clean and dry after use.
4. The suction tip and collection cup can be sterilized with boilingwater or steam, but not in a microwave.
5. The main body of the product is not waterproof, so do not rinse it with water or clean it at high temperatures. Please use a soft, damp cloth to wipe the main body for cleaning.
6. Avoid storing the product in direct sunlight or in a humid environment.

TROUBLESHOOTING

To ensure the optimal performance and longevity of your baby's nasal aspirator, it's essential to address common concerns and know how to properly maintain the device. Here are some frequent issues and solutions:

Common Issues	Solutions
The nasal aspirator not turning on or turning off after a few flashes.	Charge the nasal aspirator fully before use as it may have a low battery level when received. Please charge before use.
Suction power is low or the nasal suction effect is unsatisfactory.	1. Check if all parts are properly installed. 2. The nasal aspirator uses airflow to suction nasal mucus. During operation, avoid completely sealing the suction nozzle against the skin, as it may create a closed environment and result in poor suction or unsatisfactory nasal suction effect.
Difficulty in cleaning dried or hardened nasal mucus.	Dried or hardened nasal mucus has strong adhesion. Soften it by using a saline solution before attempting to clean it.

Nasal mucus spraying out from the air outlet during suction.	Please turn off the device, clean the collection cup, and ensure it is not filled beyond half capacity.
--	---

TECHNICAL DATA

POWER SUPPLY	5V DC/1A
POWER	5.5 W
BATTERY	3.7V/2600mAh lithium-ion
CHARGING TIME	Approximately 3 hours
INTENSITY LEVELS	9 levels
SUCTION STRENGTH	max. 70Kpa
PRODUCT NET WEIGHT	276g
PRODUCT SIZE	105mm * 75mm * 51mm

WARRANTY

The seller is liable for defective goods for 2 years from the date of delivery.

All pictures and drawings are for reference only.

Disposal of electrical equipment

The symbol of the crossed-out wheeled waste container placed on the products informs that used electrical and electronic equipment (hereinafter also "waste equipment" or "electro-waste") may not be disposed of with municipal waste. By throwing e-waste into the container, you pose a threat to the environment. Used equipment may contain substances (e.g. lead, cadmium, chromium, bromine, mercury, freon) that are dangerous to human health and life and to the environment. By sorting and handing over used equipment for processing, recovery, recycling and disposal, you protect the environment against pollution and contamination, you contribute to reducing the use of natural resources and lowering the production costs of new equipment.

DECLARATION OF CONFORMITY

Malis B. Machoński Sp. K. declares that device described in this manual meets all the requirements of the:

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